



How to Use the FRIO Mini Pocket Sharps Container



Mini sharps container

- Holds ~ 25 pen needles
- Will not hold large outer needle cap
- The black lid slides 'back to open' and 'forward to close' (you will hear a 'click')
- When the container is full, slide the lid to the 'lock' position*

(Once **locked**, you will no longer be able to open the mini sharps)



Mini sharps container


- Open the lid
- Note that when the lid is open there is an inverted 'V' area



Mini sharps container

Removing the needle:

- Insert the pen, with the needle attached, into the opening of the container
- Place the needle as close to the 'V' as possible
- Ensure the edge or collar of the needle tip is caught under the opening

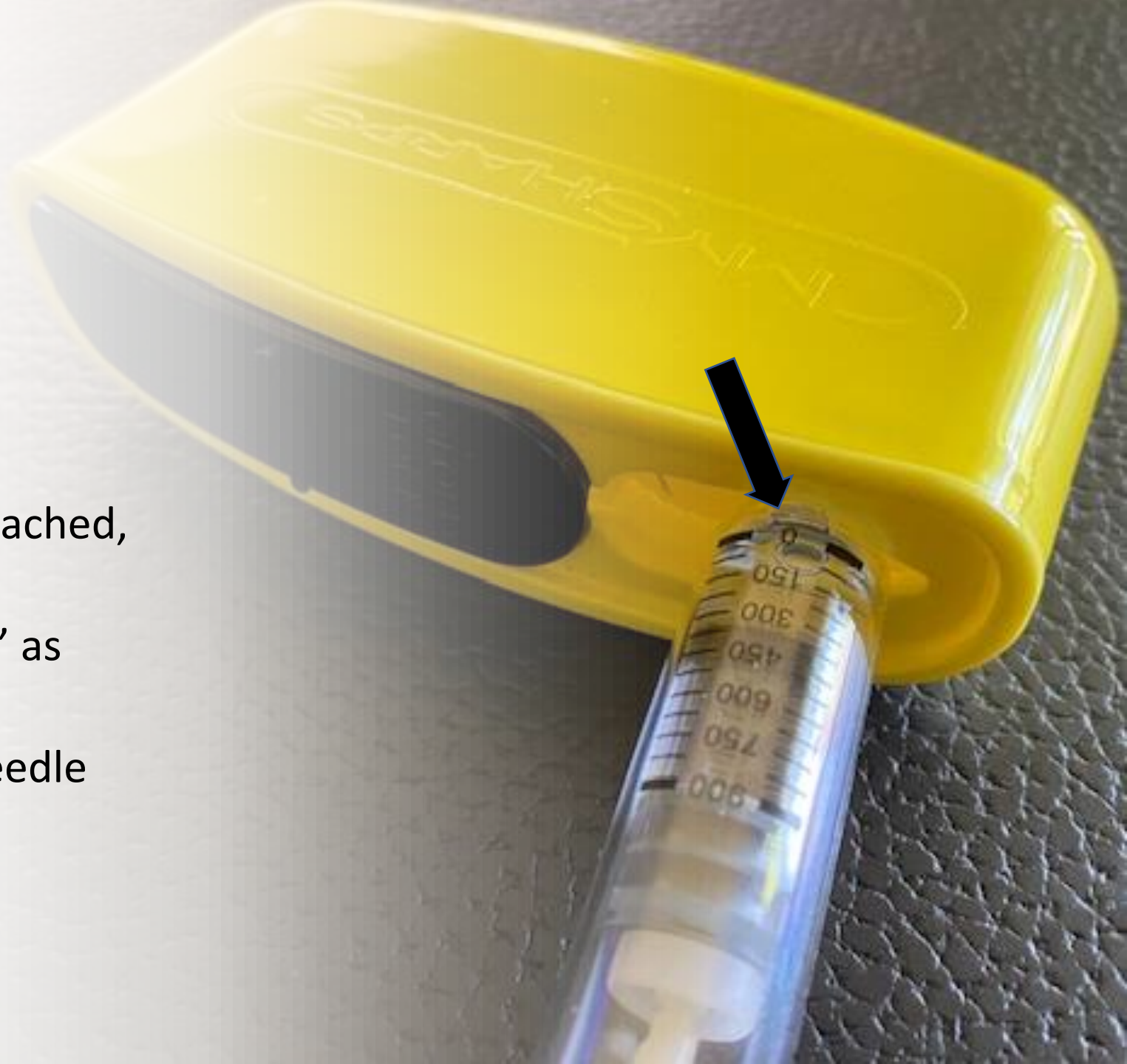

Mini sharps container

- Slide the black lid forward so that it catches the back edge of the needle collar
- Twist the pen in a counter-clockwise direction on a slight angle
- You may have to twist several times to loosen the needle from the pen
- Firmly pull the pen back and the needle should release and fall into the container





Mini sharps container

- Ensure you slide the lid to the 'Close' position after removing the needle